


# WALZ LITHOTRON EHL QUICK REFERENCE AND INFORMATION GUIDE

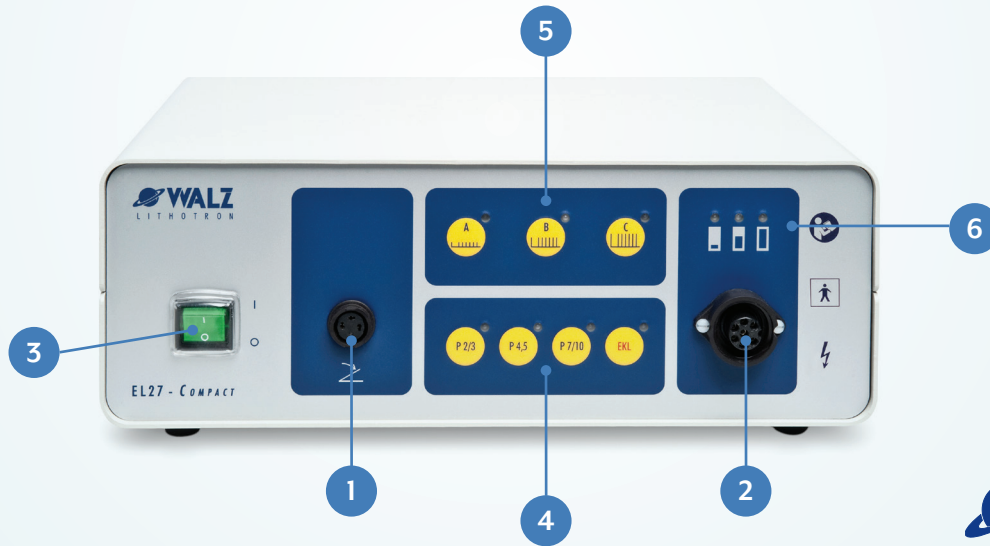

**ALWAYS USE 0.9% NaCl (NORMAL SALINE)**

1. Connect foot switch
2. Connect probe cable to generator, then connect probe cable to probe
3. Switch power on (\*never switch off during use: probe may expire)
4. Confirm probe type is automatically selected
5. Select intensity
  - a. Note: Start with intensity "A" and increase if necessary
  - b. Place probe **directly onto center of stone** while maintaining distance to tissue
6. Probe service life display
  - a. Note: For longer probe service life
    - i. Only use single pulses
    - ii. Use higher intensity with harder stones

Probe Intensity	# of Pulses per Cycle		
	P24	P30	P45
Intensity A	10	20	20
Intensity B	10	10	20
Intensity C	5	5	10

EHL Electric charge	
Intensity A	250 mj / 40Hz
Intensity B	500 mj / 40Hz
Intensity C	900 mj / 20Hz

To trigger a pulse:

Single Pulse: Tap foot pedal

Pulse Sequences: Press and hold foot pedal (pulses stop automatically after 1.5 seconds)

\*Reference Guide is not meant to substitute or replace IFU.

\*\*Please read IFU in its entirety before use.

