










Instructions for Use

	Non-Sterile		Keep dry
	Do not use if package is damaged		Keep away from sunlight
	Consult instructions for use		Do not contain DEHP
	Not made with natural rubber latex		For Single use only. Do not re-use
	Temperature limit		Federal law (U.S.A.) restricts this device to sale, distribution and use by or on the order of a physician.

IMPORTANT INFORMATION

Caution: Federal law restricts this device to sale by or on the order of a physician. Read all instructions carefully before use. They contain essential information on using this device safely and effectively. Keep these instructions in a safe, accessible location, as you may need to refer to them again. If you have any questions or comments about any information in these instructions, please contact Micro-Tech USA.

INTENDED USE

Disposable Distal Tip Protector is intended to be used on the distal tip of an endoscope for the purpose of protecting the endoscope's delicate lens and the endoscope's bending rubber to protect delicate components/mechanisms.

CONTRAINDICATIONS

- There are no known contraindications.

WARNINGS AND PRECAUTIONS

- The product is intended for single use only! DO NOT re-use, sterilize or reprocess. Re-use, sterilization or reprocessing may compromise the structural integrity of the device and/or lead to device failure. Re-use, sterilization or reprocessing may also create a risk of contamination of the device and/or cause patient infectious disease(s). Micro-Tech assumes no liability with respect to instruments reused, or reprocessed.
- Users must carefully read and follow all instructions in the operating manuals supplied with all related equipment.
- Users should wear personal protective equipment such as gloves, gowns, face masks, goggles, etc, to minimize the risk of cross contamination.
- Do not use this product for any purpose other than the stated intended use.
- Do not use the product if the package is damaged.

Product Name:

BW03-DTPL
Disposable Distal Tip Protector, Large
For Endoscopes with 7mm to 15mm diameter

BW03-DTPS
Disposable Distal Tip Protector, Small
For Endoscopes with 2mm to 10 mm diameter

Production Date: See packaging

Shelf Life: 5 years

PREPARATION

- Inspect the package before use for any damage. Do not use if package is damaged. Open the package carefully after verifying the shelf life.

- If this device shows any signs of damage, do not use. Do not attempt to repair a nonfunctional or damaged device.

INSTRUCTIONS FOR USE

1. Attach the tip protector to the distal end of the endoscope, after the endoscope has been properly cleaned, disinfected/sterilized, and dried
2. Transport and store the disinfected or sterilized endoscope
3. Remove the tip protector prior to use of the endoscope on a patient
4. After the procedure, attach the tip protector to the used endoscope for transportation of the endoscope to the reprocessing area
5. Remove the tip protector prior to cleaning, soaking, disinfecting, and/or sterilizing the endoscope to ensure that all surfaces of the endoscope are exposed during the selected process
Note: DO NOT reuse the dirty endoscope tip protector on a disinfected or sterilized endoscope

STORAGE

The product should be stored in a cool, dry, clean, well-ventilated, non-corrosive gas environment. Do not expose the package to organic solvent, ionizing radiation, or ultraviolet radiation.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with hospital, local and administrative laws and regulations.

Limited Warranty and Disclaimers

Limited Warranty to Buyer. Micro-Tech USA warrants to Buyer that, for the earlier of one (1) year from the date of purchase, or until the product is used by Buyer. The products will be free from defects in materials and workmanship when stored and used in accordance with the instructions for storage and use provided by Micro-Tech USA and in accordance with applicable regulatory requirements.

Descriptions or specifications appearing in Micro-Tech USA's literature are meant to generally describe the products and do not constitute any express warranties.

In the event that Micro-Tech USA gives technical advice with respect to the product, it is agreed that such advice is given without any liability on Micro-Tech USA's part. Any guarantee of specific properties of or in the products shall only be effective if and to the extent specifically confirmed by Micro-Tech USA in writing. These warranties shall not apply for product failure or deficiency due to improper storage, alteration, or the consequences of uses for which the products were not designed or that adversely affect the products' integrity, reliability, or performance.

Disclaimer and Release. THE WARRANTIES, OBLIGATIONS, AND LIABILITIES OF Micro-Tech USA AS SET FORTH HEREIN ARE EXCLUSIVE. BUYER HEREBY WAIVES ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO THE PRODUCTS AND ANY OTHER GOODS OR SERVICES DELIVERED BY BUYER, INCLUDING, BUT NOT LIMITED TO: (1) ALL OTHER EXPRESS AND IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND (2)

ANY IMPLIED WARRANTY ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING, OR USAGE OF TRADE.

Implied Warranties. The purchase of products may be subject to laws in the territories applicable to the sale of the products by Micro-Tech USA to Buyer that impose implied warranties, conditions, or obligations that cannot be excluded, restricted, or modified, or can only be excluded, restricted, or modified to a limited extent. The provisions of Paragraphs 2 and 4 shall apply to the greatest extent allowed by such laws.

Limitation of Liability. Except to the extent prohibited by applicable law, Micro-Tech USA's liability under this warranty is limited to: (a) the replacement of the products or the resupply of equivalent products; (b) the repair of the products or payment of the cost of

repairing the products; or (c) payment of the cost of replacing the products or acquiring equivalent products. Micro-Tech USA shall have no obligation or liability, whether arising in contract (including warranty), tort (including active, passive, or imputed negligence, strict liability, or product liability) or otherwise, for any special, consequential, punitive, incidental, or indirect damages, or for loss of use, loss of revenue, loss of business, lost profit, or other financial loss arising out of or in connection with any product or other goods or services furnished by Micro-Tech USA, even if Micro-Tech USA was aware of the possibility of such damages or loss.



Manufactured for:

Micro-Tech USA Inc.
2855 Boardwalk Drive,
Ann Arbor, MI 48104 USA
Tel: 734-259-3768
Toll free: 877-552-4027
info@micro-tech-usa.com
Made in P.R. China

BW03 Rev: C